

SNF PressCeramTM

Z I R C O N I A

**Press - To - Zirconia
Layer - To - Zirconia**



SWISS NF METALS, INC.
For Canada Call: 1-800-387-5031
Phone: 416.510.2220 / Fax: 416.510.2207
WWW.SWISSNF.COM

*For More Information
Please Call...*



For USA Call: 1-800-531-2444
Phone: 714.685.0907 / Fax: 714.685.1428
WWW.MONARCHMETALSINC.COM

SNF PressCeram™

Z I R C O N I A



The System

With the increasing popularity of Zirconia-Oxide (ZrO₂), it is only natural that the SNF PressCeram™ family has been extended to include a Press-To-Zirconia component. SNF PressCeram™-Z is a system that fully utilizes the advantages of pressing to a substructure. Just wax-up over the Zirconia-Oxide framework, invest, burnout and press. Attaining accurate margins and increased profitability by pressing to Zirconia is possible with SNF PressCeram™ - Z.



The Porcelain

The SNF PressCeram™-Z layering ceramic is a low fusing (780°C) dental ceramic sinter material, used for the production of crowns and bridges made of Zirconia-Oxide based frameworks. The newly developed chemical composition of the primary materials, their structure and particle distribution, guarantees an easy, straightforward and safe application without a lengthy cooling process. The SNF PressCeram™-Z layering ceramic is available in a wide range of shades according to the Classical A-D shade system. Combining translucence and opalescence allows for aesthetic, high-quality, natural-looking results.



The Ingots

The SNF PressCeram™-Z ingots are dual-phased glass ceramic that is pressed at approximately 900°C. The ingots have a high density and low-wear to opposing dentition. All ingots are used with the Stain & Glaze Technique, the Layering Technique, or both. The ingots are fluorescent and opalescent and most are available in 5g along with the standard 2g size. Some ingot shades are also available in different levels of opacity.



Indications

The SNF PressCeram™-Z system is designed to press over any type of Zirconia-Oxide material. It is ideal for: Single Crowns, Short & Long-span Bridges, Maryland and Inlay Bridges. Although there are great advantages to Press-To-Zirconia, the SNF PressCeram™-Z porcelain can also be used exclusively as a layering porcelain.



The Liner

The Liner is a great tool to assist the technician with achieving the proper shade of a restoration. Most Zirconia-Oxide frameworks have a shade which is infiltrated in the core, and thus do not require the use of a Liner. However, if the Zirconia-Oxide framework is stark white, use the Liner for its masking properties. The Liner is NOT necessary and will not affect the bonding properties of the porcelain to Zirconia-Oxide.

Try the Test Kit Today!

The Test Kit includes:

Liner 1 (10g), Dentin A1 (10g), Opaceous Dentin A1 (10g), Incisal 1 (10g), Translucent Light Blue (10g), Clear (10g), Glaze Powder (5g), Universal Glaze Liquid (20ml), Modelling Liquid (20ml)


SWISS NF METALS INC
For Canada Call: 1-800-387-5031
Phone: 416.510.2220 / Fax: 416.510.2207
WWW.SWISSNF.COM

*For More Information
Please Call...*


For USA Call: 1-800-531-2444
Phone: 714.685.0907 / Fax: 714.685.1428
WWW.MONARCHMETALSINC.COM